



C/ Santa Anna nº 5  
03801 Alcoy (Alicante) Spain  
tel. 606963128 y 650782761  
<http://www.primlab.com/>

07/27/2020 - V.2

## TECHNICAL DATA SHEET TNT REUSABLE HYGIENIC MASK - 4 LAYERS

<b>Description:</b>	Mask with 4 layers of TNT fabric.
<b>Composition:</b>	Outer part: 2 layers, 100% Hydrophobic Polypropylene, 80g / m2. Middle part: 2 layers, 80% Polyester / 20% Viscose, 88g / m2.
<b>Grammage:</b>	Outer part: 2 layers, 80g / m2. Middle part: 2 layers, 88g / m2.

### DESCRIPTION CHARACTERISTICS FABRIC

CHARACTERISTICS	SPECIFICATION	METHOD
Composition	73% Polypropylene 21% Polyester 6% Viscose	ISO 1833
Total Weight g / m2:	168 g / m2	EN 12127
Fabric construction	TNT - Non-Woven Fabric	BS 5441
Elastic rubber composition	83% Polyester - 17% Elastane	ISO 1833
Bacterial Filtration Efficiency (BFE) (%) (Original > 90)	90.01	EN 14683: 2019+AC: 2019
Breathability: Pressure Differential (Pa/cm2) (Original <60)	23.6	EN 14683: 2019+AC: 2019
Bacterial Filtration Efficiency (BFE) (%) (after 5 washes > 90)	97.52	EN 14683: 2019+AC: 2019
Breathability: Differential pressure (Pa / cm2) (after 5 washes <60)	17	EN 14683: 2019+AC: 2019
Report number (Aitex):	2020TM2192	
Antiviral activity	10-12% CO2pure	EN 14476: 2013+A2: 2019

UNE 0065: 2020 specification. Reusable hygienic masks for adults and children. Requirements for materials, design, preparation, marking and use.


### PACKAGE DESCRIPTION

Size	9.5X18cmAdults	Colour	White	Quantity:	300 units per box (bag of 10 uts)	Adjustable with elastic bands
------	----------------	--------	-------	-----------	--------------------------------------	-------------------------------

Expiration date: 5 years from the date of manufacture

**"WARNING: This device is not a medical device in the sense of Directive 93/42 of Regulation EU/2017/745, nor an individual protection equipment in the sense of Regulation EU / 2016/425".**

Installation instructions:

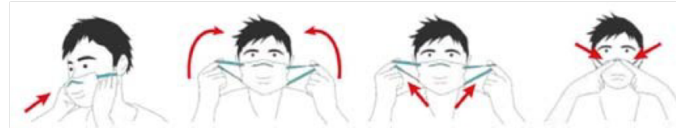

**Instructions for use:**

- Wash your hands with soap and water.
- Identify the upper part of the mask with the twist band.
- Position the mask on the face, hold the mask from the outside.
- Lower the lower part of the mask to the chin and adjust the twist band to the nose.
- To remove the mask, wash your hands with soap and water, remove the mask without touching the front.

**Washing conditions:**

- Immerse the masks in a 1:50 bleach dilution with warm water for 30 min. Then wash with soap and water and rinse well to remove any residual bleach and allow to dry.

